

# Instrument Rust and Stain Remover



**Acidic Cleaning Solution for removal of rust and stains from surgical instruments – Concentrate**

For removal of rust and stains from surgical instruments.

**Product Features:**

- Concentrated acid formula for the removal of rust and stains from metal surfaces and instruments
- Improves surgical instrument performance and appearance

**Key Formula Ingredients:**

- Phosphoric acid
- Surfactant Blend

Key Chemical Attributes	
pH Neat	<1
pH 1% Solution	0.8-1.5
Density	1.08-1.09
Color	Yellow

**Material Compatibility:**

The Belimed Protect® Instrument Rust and Stain Remover demonstrates a wide range of material compatibility and is suitable for the following materials:

- Stainless Steel
- Aluminum
- Anodized Aluminum
- Titanium
- Copper
- Brass
- All commonly used plastics and elastomers, including Polyurethane, Silicone, and Teflon
- Latexes and Synthetics (Vinyl and Nitrile)

**Handling and Storage:**

Do not store at extreme hot or cold temperatures.

**Safety Information:**

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.

**Safety Information:**

Before using product please refer to Instructions for Use, Safety Data Sheet and product label.



**DANGER:** Corrosive acid pH<2. Will cause eye damage, intestinal damage, harmful to skin. Wash hands and exposed areas thoroughly after handling. Wear eye protection and protective gloves.


**Instructions for Use – Soaking:**

1. Create dilution by combining 1 part concentrate to 10 parts water
2. Soak products for 15 minutes or until metal surface is clean
3. Periodic agitation may be used to speed up cleaning process
4. Remove items from solution and rinse thoroughly
5. Proceed with standard decontamination process

**Instructions for Use – Brush Application:**

1. Create dilution by combining 1 part concentrate to 10 parts water
2. Apply liberally to the metal surface(s), brush and let stand for 10 minutes
3. Repeat brushing and rinse instruments
4. Proceed with standard decontamination process

**Product Ordering Information:**

Product Description/ Name	Color Code	Order #	Item #	Pkg	Units Per Case
Instrument Rust and Stain Remover		106-8094	2183034	1 gal	4

REV12.22.2020\_Q0

Engineers of Confidence.